**APPROVATO**

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ARTWORK

THIS ARTWORK IS PROPERTY OF:

**SIFI**

PRODUCT NAME:

**INS Eyestil Gel mono**

PRODUCT CODE:

**1G809070P1**

COUNTRY:

**Italia/Francia/Spagna**

1° Draft: 02/08/2018

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**Italia/Francia/Spagna**

1\* Draft: 02/08/2018

ELABORATO: **Leaflet**  
 FORMULAZIONE: **Eye Drops**  
 RICHIESTA DMS N°: **311**  
 SEPARATIONS: (**Black - Fustella**)  
 BARCODE TYPE:  
 CUTTING PTOFILE:  
 MEASURES (mm): **400x220**  
 FOLDED (mm):  
 FONT SIZE: **8 pt** SPACE: **9 pt**

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### Indications

**EYESTIL® GEL** gives relief to the symptoms of ocular dryness. It is intended to soothe symptoms of ocular discomfort due to wearing contact lenses.

**EYESTIL® GEL** is also indicated for the medication of wounds and abrasions of the ocular surface caused by traumatic events and after ocular surgery.

### Composition

Sodium hyaluronate, xanthan gum, sodium chloride, potassium chloride, magnesium chloride hexahydrate, calcium chloride dihydrate, disodium phosphate dodecahydrate, monobasic sodium phosphate monohydrate, sodium citrate, glycerol, purified water.

### Presentation

30 preservative-free single-dose vials containing 0.4 ml of gel  
 1 package leaflet.

### Characteristics and mechanism of action

**EYESTIL® GEL** contains two natural polymers: sodium hyaluronate and xanthan gum. After instillation in the conjunctival sac, **EYESTIL® GEL** mixes with tears and forms a lubricating, viscous and transparent layer, reducing friction due to eye movements and blinking.

**EYESTIL® GEL** alleviates the symptoms of ocular dryness due to environmental and climatic conditions such as pollen, dust, glare, pollution, computer monitor use.

**EYESTIL® GEL** also protects the eye surface during wound or abrasion healing processes.

The absence of preservative makes the product well tolerated.

### Dosage and administration


Instil 1 drop of **EYESTIL® GEL** in the conjunctival sac 4 times daily, unless instructed otherwise by a doctor.

### Directions for use


Wash/sanitize your hands before use.


1. Make sure that single-dose container is intact.  
 2. Pull apart the single-dose container from the strip.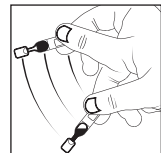

3. Hold the single-dose container from the lower part and shake it downwards.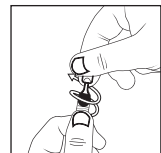

4. Open the unit by twisting and pulling the cap.

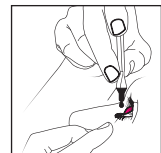5. During instillation of **EYESTIL® GEL**, avoid touching the eye or any surface with the tip of the container.

### Contraindications

Hypersensitivity to product components.

### Precautions and warnings

Do not use **EYESTIL® GEL** if the container is opened or damaged.

Avoid touching the eye or any surface with the tip of the opened container.

Each single-dose container should only be used immediately after opening. Any residual solution in the vial should not be used since **EYESTIL® GEL** does not contain preservatives and the remaining product may become contaminated, increasing the risk of eye infections.

Store below 30 °C.

The single-dose containers must be used within 12 weeks after opening of the sachet.

Do not use **EYESTIL® GEL** if it is not transparent.

Do not use **EYESTIL® GEL** after the expiry date.

Do not dispose of the product in the environment after use.

The administration of **EYESTIL® GEL** to children or people with limited skills should be supervised by a responsible adult.

### Interactions with other drugs or medical devices for ocular use

Product interactions with the administration of other ophthalmic drugs are unknown. However, due to the physical properties of the gel, it is advised not to administer other drugs or medical devices for ocular use within 60 minutes after the instillation of **EYESTIL® GEL**.

The concomitant use of therapeutic contact lenses after photorefractive keratectomy (PRK) surgery resulted to be compatible with the use of the product.

However, the use of other kinds of contact lenses is advised only 60 minutes after the application of **EYESTIL® GEL**.

### Undesirable effects

Transient and mild burning may occur occasionally.

### FOR TOPICAL OCULAR USE ONLY

**KEEP OUT OF THE SIGHT AND REACH OF CHILDREN**

### Manufacturer

SIFI S.p.A. - Via Ercole Patti, 36  
 95025 Aci S. Antonio (CT) - ITALY

**EYESTIL® GEL** is a sterile medical device.

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**SIFI S.p.A.** - Via Ercole Patti, 36