THIS ARTWORK IS PROPERTY OF:

SIFI

PRODUCT NAME:

Leaflet - Eyestil PLUS

PRODUCT CODE:

1G154330T1

COUNTRY:

Francia - West Africa

1* Draft: 06/07/2018

ELABORATO: **Leaflet**FORMULAZIONE: **Eye drops**SEPARATIONS: **(Black - Fustella)**

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MEASURES (mm): **130x360**PIEGA: **22,5**

CUTTING PROFILE:

FONT SIZE: **7 pt**SPACE: **8 pt**

FRONTE

ISSUE DATE

30-07-18

OPERATOR

G.Longo

DRAFT

08

FR

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Dernière revision: **01/2018**

EN

Eyestil PLUS**Lubricant and protective ophthalmic solution
containing sodium hyaluronate 0.4%**

Eyestil PLUS is a sterile, isotonic ophthalmic solution containing sodium hyaluronate and distilled waters of ginkgo biloba, blueberry, fennel and centella asiatica.

The solution is packaged in 10 ml multidose containers. It is preservative-free.

Sodium hyaluronate gives the product mucomimetic and visco-elastic properties that guarantee an adequate lubrication of the eye surface, protecting it from the friction generated by eye movements.

Moreover, the polymer is able to retain water, thus **Eyestil PLUS** keeps the eye surface hydrated and increases the volume of tear film, protecting the ocular mucosa from external agents.

The solution also contains boric acid-sodium tetraborate buffer as well as sodium chloride, to ensure a physiological pH.

The product is phosphate-free.

Eyestil PLUS can be applied while wearing soft, semi-rigid and rigid contact lenses, without the need to remove them.

INDICATIONS FOR USE

Eyestil PLUS protects, hydrates and lubricates the eye surface while providing relief for burning sensation, irritation and discomfort symptoms related to dry eye, and also in the following cases:

- environmental factors (air conditioning, over exposure to sun radiation, wind, smoke, pollution, severe weather conditions, dust);
- eye fatigue caused by prolonged study, work, and driving hours, and by excessive use of electronic devices;
- long-term use of soft, semi-rigid and rigid contact lenses.

INSTRUCTIONS FOR USE

- Before the first application, remove the safety seal and the protection cap;
- Instill 1 - 2 drops in each eye by gently squeezing the bottle;
- Shake the bottle down in order to remove any residual on the dropper;
- Close the bottle immediately after use.

The product must be used within 6 months after first opening.

Eyestil PLUS is also intended for frequent applications, several times a day, according to your needs or doctor's instructions.

COMPOSITION

0.4% sodium hyaluronate - distilled waters of ginkgo biloba, blueberry, fennel and centella asiatica - boric acid - sodium tetraborate - sodium chloride - water for injection.

It is preservative-free.

It is phosphate-free.

1G154330T1

THIS ARTWORK IS PROPERTY OF:

SIFI

PRODUCT NAME:

Leaflet - Eystil PLUS

PRODUCT CODE:

1G154330T1

COUNTRY:

Francia - West Africa

1* Draft: 06/07/2018

ELABORATO: **Leaflet**FORMULAZIONE: **Eye drops**SEPARATIONS: **(Black - Fustella)**

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MEASURES (mm): **130x360**PIEGA: **22,5**

CUTTING PROFILE:

FONT SIZE: **7 pt**SPACE: **8 pt**

RETRO

ISSUE DATE

30-07-18

OPERATOR

G.Longo

DRAFT

08**APPROVATO****Da ecollazo alle 11.52 , 31/07/2018****WARNINGS AND PRECAUTIONS**

- The product is for ophthalmic use only.
- Do not use the product if the bottle is damaged or not perfectly sealed when opening for the first time.
- During application, ensure that the tip of the bottle does not come into contact with the patient's eye or eyelids or any other surface.
- Do not use in the case of a known incompatibility with any component of the formulation.
- In case of side effects, stop using the product and consult a doctor.
- Do not use the product after the expiry date indicated on the packaging. The expiry date refers to the properly stored, unopened product.
- Keep out of reach of children.
- Do not dispose of the product in the environment after use.

SIDE EFFECTS

No undesirable effects are known to-date.

PACKAGING

Preservative-free 10 ml bottle.

STORAGE

Store below 30° C.

SHELF LIFE

The unopened product must not be used after the expiry date printed on the packaging.
The product must be used within 6 months after first opening.

**SIFI S.p.A.**

Via Ercole Patti, 36
95025 Aci S. Antonio (CT)
ITALY

Eystil PLUS is a sterile medical device.

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